

DE


Montage des Wandhalters

Hinweis LIFE-BASE II kann ausschließlich in diesen Wandhalter eingehängt werden.

1. Suchen Sie einen geeigneten Befestigungsplatz, der **planeben** und **senkrecht** ist.
2. Markieren Sie alle 7 Bohrungen an der Befestigungsstelle.
3. Bohren Sie die Befestigungslöcher mit einem Bohrer \varnothing 3 mm.
4. Schrauben Sie den Wandhalter mit den beiliegenden 7 Schrauben an.

Die Trageplatte muss waagrecht montiert werden.

Wichtig: Der Befestigungsuntergrund muss eine ausreichende Festigkeit aufweisen, um die Wandhalterung und die Trageplatte nach DIN EN 1789 Krankenkraftwagen sicher zu halten.

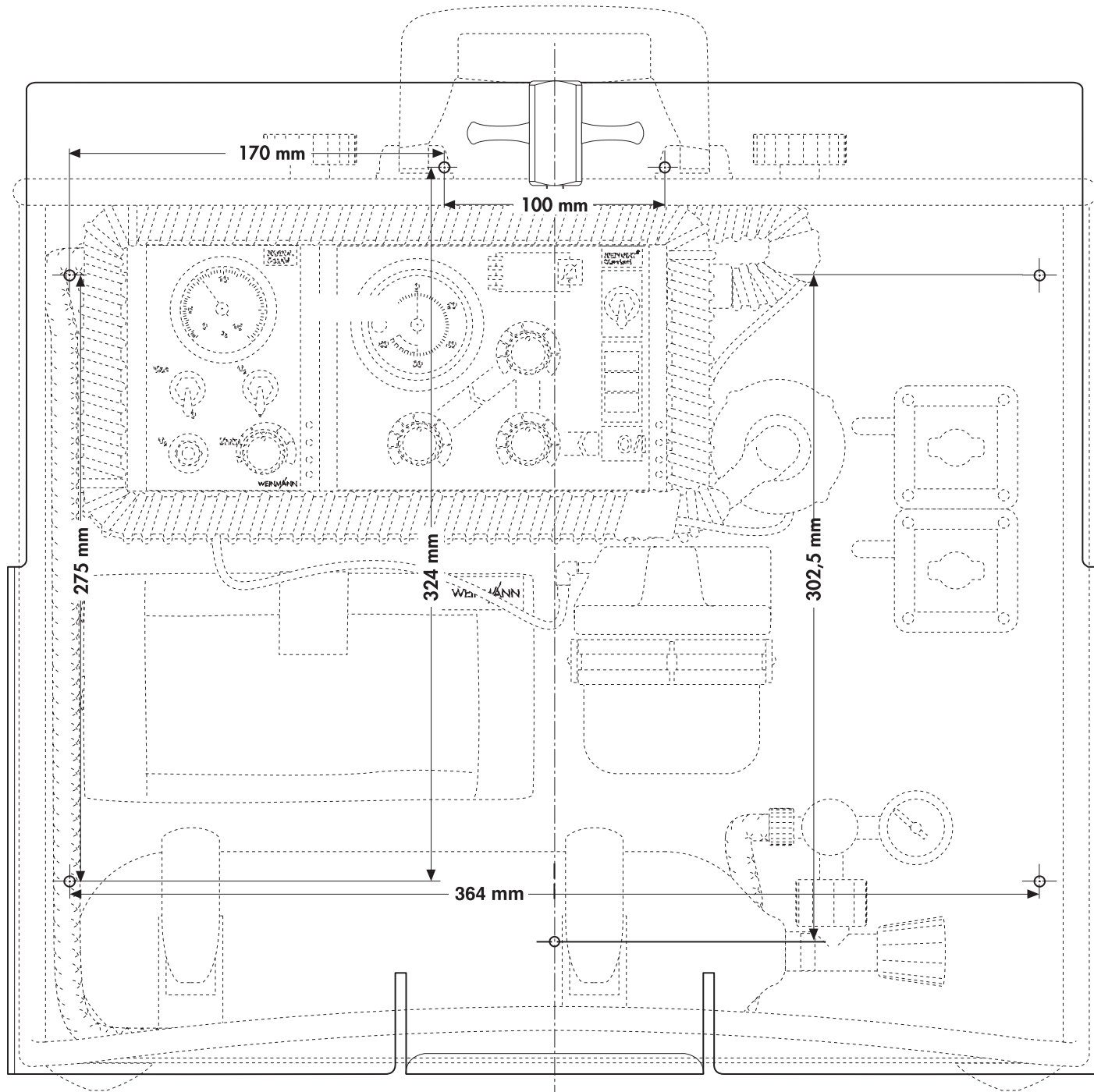

EN

Fitting the wall bracket

Note LIFE-BASE II may only be suspended from this wall bracket.

1. Look for a suitable place for mounting the bracket, which is **plane** and **vertical**.
2. Mark all 7 drill-holes on the place for fitting.
3. Drill the holes using a 3-mm drill bit.
4. Screw on the wall bracket using the 7 screws provided.

The bearing plate must be fitted level.

Important: the surface to which the appliance is being attached must be sufficiently firm for it to hold the wall bracket and the bearing plate securely in accordance with DIN EN 1789, the standard covering ambulances.

FR

Montage du support mural

Remarque Le LIFE-BASE II ne peut être suspendu qu'à l'aide de ce support mural.

1. Chercher un endroit **plat** et **d'aplomb** convenant pour la fixation.
2. Marquer l'emplacement de l'ensemble des 7 trous à percer à l'endroit choisi.
3. Percer les trous avec une mèche de \varnothing 3 mm.
4. Visser le support mural en utilisant les 7 vis fournies à cet effet.

La plaque de montage doit être montée dans la balance.

Important : la surface de fixation doit être suffisamment rigide pour pouvoir supporter le support mural et la plaque de montage selon la norme DIN EN 1789 sur les chariots hospitaliers.

BASE STATION II

Wandhalterung für LIFE-BASE II /
Wall bracket for LIFE-BASE II /
Support mural pour LIFE-BASE II

Montageanleitung

Assembly instructions


Notice de montage

Manufacturer
WEINMANN Emergency
Medical Technology GmbH + Co. KG
Frohbosestraße 12
22525 Hamburg
GERMANY
T: +49 40 88 18 96-120
E: customerservice@weinmann-emt.de

Center for Production, Logistics, Service
WEINMANN Emergency
Medical Technology GmbH + Co. KG
Siebenstücken 14
24558 Henstedt-Ulzburg
GERMANY

WM 16213e 03/2016 DE, EN, FR

 Simply Professional

 Made in Germany

 WEINMANN-Emergency.com

